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UNSERE MISSION

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Products just for you –
that's mia


OUR MISSION

Period products just for you. So you can live your period the way you want to – with products that fit you the way they should.

Mini	●●	6–9g
Normal	●●●	9–12g
Super	●●●●	12–15g

STEP-BY-STEP INSTRUCTIONS

Whether you're a period newcomer or an experienced tampon user, your period is and always will be something special. To make sure you are well prepared for all future periods and for using mia tampons in every situation, we've put together this step-by-step guide: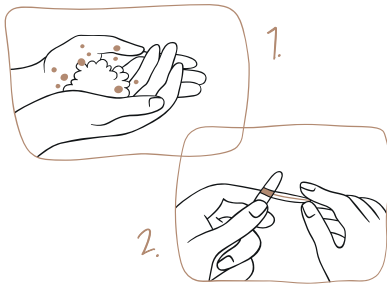


1. Getting started

Wash your hands thoroughly before and after inserting a mia tampon to prevent possible infection.

2. Opening a mia tampon

Before you open a mia tampon, please check that the packaging is intact. If this is not the case, use a mia tampon with undamaged packaging. To open, simply pull on the small colored plastic strip and remove both ends of the transparent packaging.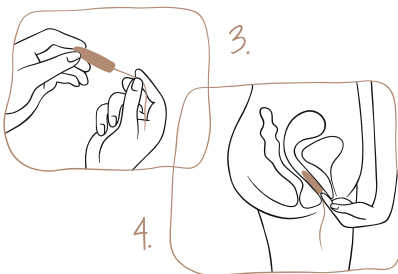


3. Checking the removal string

Unwind the cord, pull on it, and check if it is well attached to the tampon.

4. Inserting a mia tampon

To insert your mia tampon, press your index finger into the recess at the end. Take a comfortable and relaxed position, e.g. standing with slightly bent legs

or squatting. Insert your mia tampon diagonally – if you feel any resistance, change direction slightly.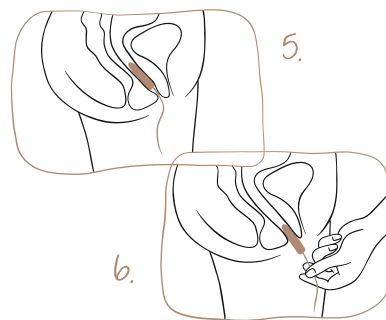

5. Positioning a tampon comfortably

Your mia tampon is positioned correctly when you can no longer feel it and the removal string is hanging from your vagina. If you can still feel your mia tampon, it is not positioned properly yet. You can then either remove it or insert it deeper.

6. Removing a mia tampon

Gently tug on the removal string and dispose of your mia tampon in the regular household waste. If you can't grab hold of the removal string, don't panic, wait until your mia tampon is soaked (e.g. you find blood in your underwear) and start pushing it out with the power of your muscles the next time you have to use the bathroom. You should then be able to remove it more easily with two fingers.

TSS INFORMATION

Toxic Shock Syndrome (TSS) is a very rare but life-threatening disease that can occur in men, women, and children. It is caused by toxins of a certain bacterium often found in humans on the skin, in the nose, or in the vagina. The risk of developing TSS is higher in women under 30 years of age and is associated with the use of tampons but can occur regardless of your period. **Symptoms** – If during your period you suddenly experience a high fever (> 39°C), a sore throat, dizziness, nausea and vomiting,

diarrhea, fainting, muscle pain, or a sunburn-like skin irritation, remove the tampon immediately and consult a doctor or hospital. Mention TSS, the fact that you are on your period, and that you have been using tampons. Ask your doctor for advice before continuing to use tampons if you have had TSS in the past. **TSS can be life threatening and must be detected and treated early.**

PLEASE NOTE

If you suffer from allergies – Our mia tampons are hypoallergenic and respect your body's natural PH value, so they are suitable for anyone suffering from allergies. **Recommended duration of wear** – A tampon should be changed every 4–8 hours and never stay in your body for more than 8 hours to minimize the risk of developing TSS. **The appropriate size** – Always try to use the lowest level of absorbency when choosing a tampon for your period, so that your tampon is never worn for more than 8 hours. **The right number** – Always make sure there is only one tampon in your body, so remove the old tampon before inserting a new one. **Removal** – If you have trouble removing a tampon by yourself, please consult a gynecologist. **Correct disposal** – Please do not flush your mia tampon down the toilet but throw it into the household waste.

QUESTIONS

Do you have further questions about tampons or mia? Then we look forward to hearing from you:

LILLYDOO GMBH

Hanauer Landstraße 147-149
60314 Frankfurt am Main · Germany
service@mia-care.com
www.mia-care.com

Follow us on 